Sperm Separation Device Model: ICF

Manufactured by:

TabaLab
Tarahan Behboud Abzar (TABA)
No. 78, Italia Street, 2nd Floor
Tehran, Iran
www.microferta.com

Intended Use


Preparation of motile sperm from semen for use in assisted reproductive technology (ART) procedures, including ICSI, IVF, & IUI.

⚠ WARNING

- Discard if packaging is damaged.
- Do not clean device.
- Single-use device.
- Do not incubate more than 30 min.
- Keep the device level during use.

IMED
National Medical
Device Directorate IR.IRAN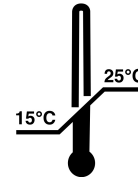

STERILE R


**Single
Use
ONLY**

Materials Required

- Micro Ferta® ICF device
- Sperm washing solution: bicarbonate- or HEPES-buffered media supplemented with 2-10% protein
- Incubator at 37 °C
- Petri dish (>90 mm)
- 1 mL Luer-lock syringes
- Sperm-safe culture tube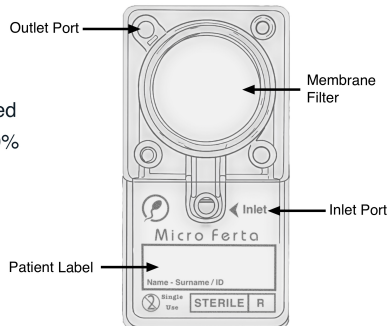

Device Description


Micro Ferta® ICF is a sterile, single-use sperm separation device that enables selection of motile sperm through a micro-structured filtration membrane without centrifugation.

SAMPLE LOADING

- Secure the device and gently insert the syringe tip into the Inlet Port, applying light pressure until a stable connection is achieved.

- Inject the sample slowly and evenly into the device.


Note: Take care to prevent air bubbles from forming beneath the membrane.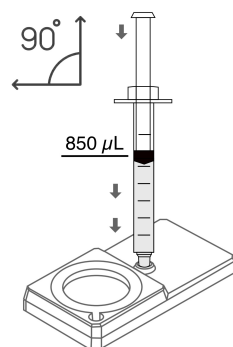

MEDIA ADDITION

- Fill a new syringe with 750 µL of culture medium.


- Prime the Outlet Port / concentration chamber by injecting approximately 50 µL of medium, until fluid reaches the membrane.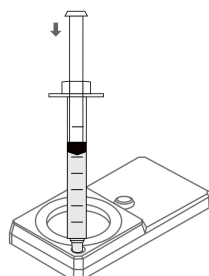

User Manual

PREPARATION

- Prepare all required materials and perform the procedure on a clean laboratory surface.
- Allow the semen specimen to liquefy by incubating it at 37 °C for 20–30 minutes.
- Open the device packaging carefully, ensuring that the membrane surface is not touched.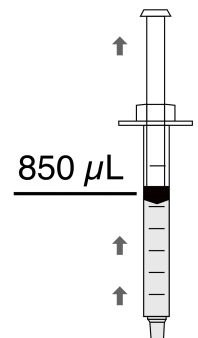

SAMPLE ASPIRATION


- Using a 1 mL syringe, slowly aspirate 850 µL of the liquefied semen sample.

Note: If the available volume is insufficient, supplement with an appropriate culture medium to reach a final volume of 850 µL.

- Detach the syringe from the Outlet Port and dispense the remaining medium onto the upper membrane surface from a height of approximately 2 cm. Ensure the membrane is fully covered and that the added medium connects with the priming droplet at the outlet channel.

Note: Do not tilt or rock the device to distribute the medium.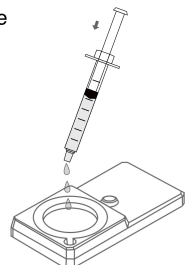


INCUBATION

- Place the device horizontally inside a petri dish and cover it. Incubate at 37 °C for 30 minutes, keeping the device level and undisturbed throughout the incubation period.

COLLECTION OF SEPARATED SPERM

- Attach a new 1 mL syringe to the Outlet Port, ensuring a secure connection. Slowly withdraw up to 500 µL of the sperm-containing fraction.
- Transfer the collected sperm fraction to an appropriate culture vessel suitable for the intended assisted reproduction application.

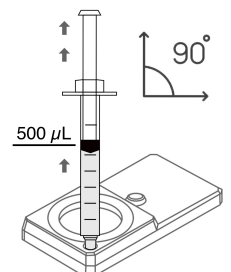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